

ApexBio
Taiwan

五鼎生技



THE EDGE
Quick Reference Guide



1) Check the Code Card

- There is a code card packaged in each box of test strips
- Verify the code number on the code card matches the one on tube package



2) Insert the Code Card

Insert the code card with the code number facing up completely into the code card port as the meter is off

ApexBio
Taiwan



3) Take out a Test Strip

- Make sure the code card is inserted every time before testing
- **Seal the tube immediately after taking out the strip**
- Strips will be expired in two months since the tube is opened



4) Insert the Test Strip to Turn on the Meter

Insert the strip as the meter is off and it will turn on automatically

ApexBio
T a i w a n



5) Check the Code Number

Check if the code number appears on the screen matches with the number on the tube package



6) Obtain a Drop of Blood


- Lance the finger
- Form 3 μ l of blood sample

ApexBio
Taiwan



7) Apply the Blood to the Test Strip

- Apply the blood to the absorbent area of the strip
- the meter will start analyzing as sufficient sample is absorbed

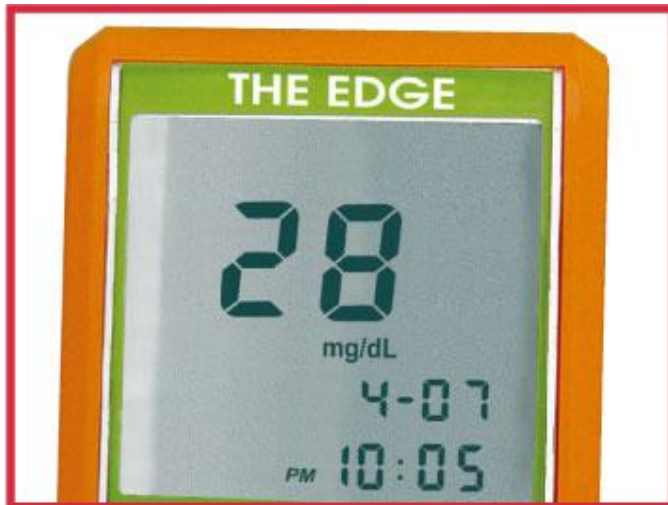

8) Get Reading

- The screen will show timing bars
- The result is displayed about 45 seconds later

ApexBio
Taiwan